

HALCO

FACE SHIELD

INFORMATION &

INSTRUCTIONS FOR USE



Usage

The Face Shield is intended to be worn by healthcare professionals to protect the face from liquid splashes and bodily fluids when treating infectious patients. It is not designed to be used as a respiratory device or to prevent any airborne infectious materials entering through the nose or mouth. The face shield does not offer any protection against mechanical impacts.

Always check the product for defects before use.

It is highly recommended that safety glasses/goggles and a face mask are used with the Face Shield when appropriate to further reduce the chance of liquid splashes and bodily fluids causing harm and add another level of protection against respiratory infections or diseases which may be present.

Maintenance and Servicing

The face shields are recommended to be used for short to mid-term periods (6-72+hours). The usage is dependent on application and user care and maintenance. See usage guidance below:

- NHS healthcare setting 6-8 hours. They should be disposed with medical waste after each usage period of 8 hours.
- Clinical medium risk 24-48 hours
- Retail low risk 48-72 hours

Storage

To be stored at room temperature.

HALCO Face Shield


Model No: HFS01

Regulation: PPE Category II of REGULATION (EU) 2016/425


Standard: BS EN 166:2001



FITTING INSTRUCTIONS

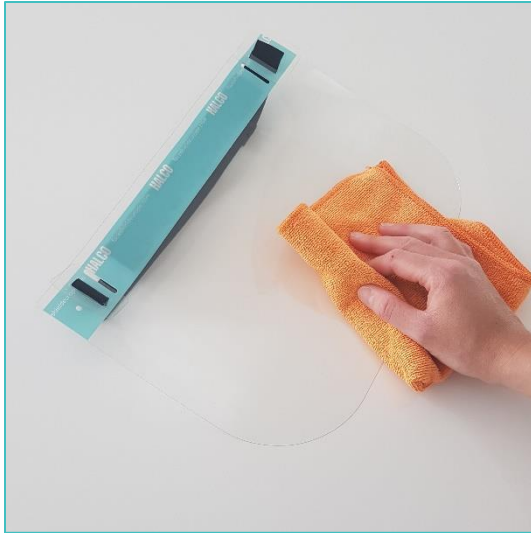

1. Remove the shield from the packaging. Wipe down the shield before use.

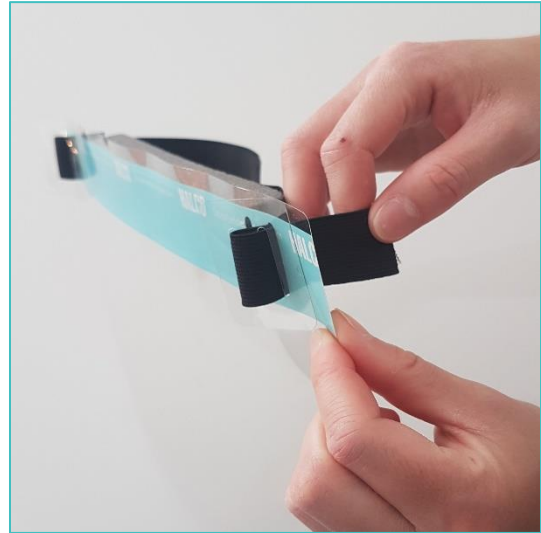

2. Adjust the strap length to ensure best fit for personal comfort



3. Place the foam cushion on your forehead and pull the elastic strap over your head



4. Done! Your shield is ready to use.